<https://www.hippodiabetes.com/downloadsandguides>

# HiPPO

BIOSENSORS

**HIPPO MEDICAL PTE. LTD.**

60 PAYA LEBAR ROAD  
#11-53  
PAYA LEBAR SQUARE  
SINGAPORE (409051)

info@hippo-biosensors.com  
<https://www.hippodiabetes.com/>

**PureCE B.V.**

Distributieweg 13  
2742 RB Waddinxveen, THE NETHERLANDS

# CE 0051

HiPPO BIOSENSORS is a registered trademark of HIPPO MEDICAL PTE. LTD. in Singapore and/or other countries.

All other product or company names that may be mentioned in this publication are tradenames, trademarks or registered trademarks of their respective owners.

## Install APP & Login

Scan the QR code to download the APP, and register a HIPPO account in the APP.

### 1. Clean the wearing site

1.1 Thoroughly wash your hands with soap and allow them to air dry.

1.2 We recommend wearing it on the outer back of the upper arm, as this area typically has more fatty tissue.

*\*The sensor isn't tested or approved for other sites.*

1.3 Disinfect the area with alcohol, wipe thoroughly, and allow it to air dry completely.

### 2. Wear the sensor

2.1 Slide the blue button to the top position.

*\*Slide the button upward until it reaches its limit, and cannot to be pushed any further, and you will see a silver sealing pop out.*

2.2 Remove the sealing ring, and then open the top cap. (You will see a button underneath the cap)

2.3 Press the button

*\*Press down until you hear a "click" sound.*

2.4 Unscrew the bottom cap

*\*Please do not press down on the applicator until it is placed on the intended site.*

2.5 Position the bottom of the applicator against the intended site. Apply firm pressure with the top cap closed, and continue pressing the applicator until you hear a distinct 'click' sound.

2.6 Gently remove the applicator vertically.

**Please do not discard the applicator now!**

2.7 Ensure the sensor is securely attached to your skin by gently rubbing the adhesive tape around it for 10 seconds.

### 3. Connect your sensor in the APP

3.1 By scanning the QR code on the back of the applicator to pair your sensor in the APP.

3.2 If you cannot scan the code, you can also input the connecting code manually.

### 4. Use the overpatch

You must apply the overpatch to keep the sensor on your body. It's inside the envelope within the sensor box.

**\*\*review the product instructions that come with your continuous glucose monitoring device. These instructions have important information, including indications, contraindications, warnings, and precautions.**

Note: You can go to  
[<https://www.hippodiabetes.com/downloadsandguides>]  
or scan the QR code on the bottom of the instructions.  
Or contact your local representative sales to get product instructions.

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